


FLTR™

General Use Face Mask

USER INSTRUCTIONS

FPK1068rL

Thank you for your purchase!

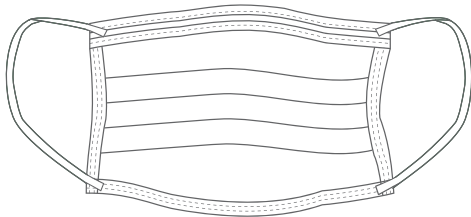

THESE MASKS ARE NOT PERSONAL PROTECTIVE EQUIPMENT AND ARE NOT INTENDED AS REPLACEMENTS OR SUBSTITUTES FOR PERSONAL PROTECTIVE EQUIPMENT. THESE PRODUCTS ARE NOT INTENDED FOR MEDICAL USE OR TO PREVENT ANY DISEASE OR ILLNESS. EACH MASK IS INTENDED FOR SINGLE USE ONLY – DISCARD IMMEDIATELY AFTER USE

Questions? Visit support.fltr.com for FAQs, additional product info and customer service requests

FLTR, Inc. FLTR.com

TOLL FREE: 1-888-525-FLTR (3587)

Hours: Mon-Fri 8am - 5pm PST

110 110th Ave NE – Suite 520 Bellevue, WA 98004

Made in China

FEATURES:

- 3-Layer Construction
- Adjustable Nose Piece and Elastic Ear Loops for Optimal Fit
- Lightweight and Disposable

PROPER FIT INSTRUCTIONS:


For proper use, follow these steps each time face mask is worn.

1. Locate the front of the mask (elastic straps are attached to the front of the mask). Be sure the front of the mask is facing out during use. (Fig. 1)
2. Elastic straps should be pre-stretched before placing face mask on the face (Fig. 2).
3. Cup the face mask in your hand, with the face mask cupped in your hand, allowing elastic straps to hang freely (Fig. 3).
4. Align face mask under chin with the nose covering area facing up. Pull each strap over your ears, allowing it to rest behind each ear. (Fig. 4).
5. Use both hands to align nose covering area over nose and adjust the nose piece to fit the shape of your face comfortably (Fig. 5). **ALERT!**
Not properly securing the nose piece by only using one hand may result in improper fit, decreased filtration, and face mask contamination.
Use two hands to secure nose piece.
6. Make sure the mask is completely secure. Make sure it covers your nose and mouth so that the bottom edge is under your chin. (Fig. 6).


IMPORTANT - RETAIN THESE USER INSTRUCTIONS:

Users must read and understand these User Instructions BEFORE USE. Retain these User Instructions for reference.


FIT INSTRUCTIONS:

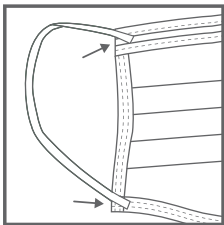


(Fig. 1)

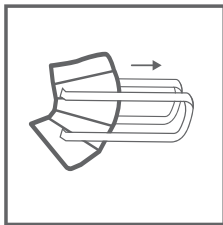


(Fig. 2)

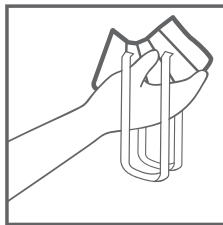

(Fig. 3)

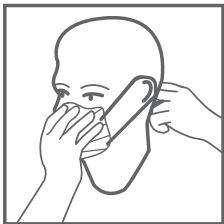

(Fig. 4)

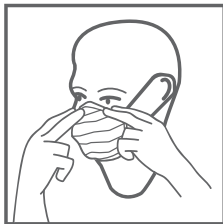

(Fig. 5)



(Fig. 6)