**ENGLISH**

### **Wrist Blood Pressure Monitor**

Instructions for use ..... 3

**ESPAÑOL**

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Manual de instrucciones..... 26

**FRANÇAIS**

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Mode d'emploi ..... 51

For Canada only/Solo para Canadá/  
Pour le Canada seulement:  
Manufacturer/ Fabricante/ Fabricant:  
Beurer GmbH, Söflinger Str. 218, 89077 Ulm Germany

Distributed by/Distribuido por/Distribué par:  
Beurer North America LP, 1 Oakwood Boulevard,  
Suite 255, Hollywood, FL 33020, USA  
www.beurer.com

**Questions or comments?**

**Call our US-based customer service toll free at  
1-800-536-0366.**

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**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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# ENGLISH

## Contents

|  |    |   |    |
|--|----|---|----|
| 1. IMPORTANT SAFETY NOTES .....          | 4  | 8. Displaying and Deleting Measurements ..... | 17 |
| 2. Getting to Know your Instrument ..... | 12 | 9. Care, Maintenance and Disposal .....       | 18 |
| 3. Package Contents .....                | 12 | 10. Troubleshooting Guide .....               | 18 |
| 4. Parts and Controls .....              | 12 | 11. Technical Specification .....             | 18 |
| 5. Preparing for Measurement .....       | 13 | 12. FCC Compliance Information .....          | 20 |
| 6. Measuring Blood Pressure .....        | 14 | 13. Warranty .....                            | 21 |
| 7. Evaluating Results .....              | 16 | 14. ELECTROMAGNETIC COMPATIBILITY INFORMATION | 23 |

## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

**If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. Please see warranty for service contact.**

### Dear Valued Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating,

weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips
- new product information
- continual outstanding customer service and support.

Visit [www.registerbeurer.com](http://www.registerbeurer.com) or contact customer service at **1-800-536-0366**.

Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer Team

## 1. IMPORTANT SAFETY NOTES

### Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

**READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

- This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.

**WARNING** WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

**NOTICE** NOTICE addresses practices not related to personal injury, such as product and/or property damage.

Follow Instructions for use

Type BF, applied part (cuff)

|   |  |
|---|--|
|   | Direct current   |
| Storage/Transport<br>  | Permissible storage and transport temperature and humidity                                     |
| Operating<br>          | Permissible operating temperature and humidity   |
| <b>IP 22</b>  | Device protected against foreign objects $\geq 12.5$ mm and against water dripping at an angle |
|    | Serial number  |

**WARNING - To reduce the risk of fire, electric shock, or serious personal injury:**

- Consult your physician or pharmacist before starting to measure your blood pressure.
- Read and understand all instructions and warnings before using this device.
- Like any oscillometric blood pressure measurement devices, certain medical conditions can affect the measurement accuracy, among others:
  - disorder of the cardiac rhythm
  - very low blood pressure
  - patients in shock

- patients with very low blood perfusion
- diabetes
- vessel anomalies
- people with electrical implants, such as a cardiac pacemaker
- women who are pregnant.

- Due to their condition the oscillometric measurement method can produce incorrect readings. This represents a risk for your health, since values may be interpreted incorrectly. Always consult your physician to determine what will be suitable for you.
- Measurements are for your information only – they are no substitute for a medical examination. Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).
- Only a physician or a trained health care professional who is familiar with your medical history is able to accurately interpret your blood pressure measurements. Consult your physician before starting blood pressure monitoring.
- This product is not intended to serve as a substitute for the advice of a physician or medical professional. This product is not intended to substitute for regular medical checkups. Contact your physician for specific information about your blood pressure.

- Please note that technically related measuring tolerances are possible. Please see section '13. Technical Specifications' for details. NOTE: A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at the same time each day for consistency.
- Any cuff related blood pressure measurement in high repetition rates, can lead to severe measurement side effects, e.g.
  - any nerve compression with temporary arm/hand paralysis
  - the release of an arterial or venous thrombus, which can cause a life threatening situation.
- Please contact your physician about the specific risks of cuff pressure in your specific case.
- The 'irregular heartbeat' function does not replace a cardiac examination, but may help to detect potential pulse irregularities at an early stage. Always consult your physician to determine what will be suitable for you.
- The 'irregular heartbeat' function is not designed for diagnosing or treating an arrhythmic disorder. Arrhythmia can only be ascertained by a licensed physician.
- The WHO chart is not intended to replace a medical diagnosis. This chart is only a reference for different classifications of blood pressure.
- If you notice abnormal or suspicious variations in blood pressure measurements, consult your physician immediately.
- Do not use the cuff on people who have undergone a mastectomy.
- Prior to use, you have to ensure that the blood pressure monitor is free of damage. If in doubt, do not use the unit and contact customer service at 1-800-536-0366.
- This device is intended only for personal, non-commercial, adult use in measuring blood pressure and pulse rate. Do not use the product for any other purpose. This product is not intended for use in a hospital, physician's office, or any other health care facility.
- The unit must be used in accordance to the specified ambient conditions, otherwise the accuracy of readings might be affected. See section 'Care, Maintenance and Disposal' and section 'Technical Specification' for details
- Place the cuff on your wrist only. Do not place the cuff on other parts of the body.

- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Please do not share the cuff with other persons, to avoid cross infection.
- Use this monitor only on humans.
- This device is not intended for use by people (including children) with restricted physical, sensory, or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.
- This product is not a toy. Keep it out of reach of children, toddlers, and infants.
- Keep the product out of the reach of pets.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.
- This product contains small parts that may present a choking hazard to children. Keep the unit and all parts out of reach of children. **NEVER LEAVE CHILDREN OR THOSE WHO REQUIRE CLOSE SUPERVISION UNATTENDED WITH THIS DEVICE.**
- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. Consult your doctor before using the blood pressure monitor during pregnancy.
- Only use the device on people who have the specified wrist measurement for the device.
- Do not use any cuffs, parts and accessories other than those explicitly recommended by the manufacturer for use with this product. Parts and accessories not approved for use with the device may cause damage to your health and to the product.
- Electromagnetic interference: Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave ovens) while it is in operation, as inaccurate measurements may result. To prevent such interference, use the unit at a sufficient distance from such devices or turn the devices off.
- Do not measure your blood pressure while operating a vehicle or in any situation which requires your full attention.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the wrist.

- Avoid any mechanical restriction, compression or bending of the cuff.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Make sure the cuff is not placed on the wrist of an arm where the arteries or veins are undergoing medical treatment, e.g. intravascular access or intravascular or therapy, or an arteriovenous (AV) shunt.
- Consult your physician if you have any doubt using the blood pressure monitor simultaneously with other monitoring medical equipments on the same limb.

**CAUTION – To reduce the risk of personal injury or product/ property damage:**

- For consistency, always measure your blood pressure at the same time each day.
- The device is not intended for use on neonates, children or pregnant women. (Clinical testing has not been conducted on neonates, children or pregnant women.)
- Before each measurement, relax for about five minutes.
- When performing multiple measurements on an individual, wait one minute between each measurement.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the results.
- In the case of restricted circulation on the arm as a result of chronic or acute vascular diseases (including vascular constriction), the accuracy of the wrist measurement is limited. In this case you should use an upper arm blood pressure monitor instead
- Do not place the cuff over wounds as this may cause further injury.
- Remove any kind of wrist jewelry or the like before taking a measurement. This could cause bruises.
- Do not place the cuff over heavy clothing (e.g. a jacket or sweater sleeve) as the blood pressure monitor will not be able to take a proper measurement and there is an elevated danger of acquiring hematoma or skin marks during the course of the measurement.
- In case the cuff does not stop inflating, interrupt the measurement by pressing the start/stop button and open the cuff at once.

- Do not disassemble the unit, it may result in injuries.
- When applying the cuff, make sure there are no wrinkles in the cuff. This could cause bruises.
- If you are allergic to plastic/rubber, please don't use this device.
- Blood pressure measurements can lead to temporary marks on the skin at the site of the cuff placement. This is especially the case in high repetition rates, in hypertonic patients and in patients with weak pulses. In rare cases a mark may persist for a couple of days. Please contact your physician about these specific risks of cuff pressure in your specific case.
- For hygienic reasons the cuff is intended only for the use by one person.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- Prolonged over-inflation (cuff pressure exceeds 300 mmHg or maintained above 15mmHg for longer than 3 minutes) of the bladder may cause ecchymoma of your arm.
- Please note that when inflating, the functions of the limb in question may be impaired.
- The accuracy of any blood pressure measurement with this device can be affected by a multitude of causes. Some can be avoided some have to be accepted at the time of measurement. Speak with your physician about possible causes affecting your blood pressure.
- Do not inflate the cuff unless wrapped around wrist.
- Do not touch the unit and cuff and/or press any buttons when measurement is in progress, except for stopping the measurement. Doing so may cause incorrect readings.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions

or a decrease in the device's electromagnetic immunity; this can result in faulty operation.

- Failure to comply with the above can impair the performance of the device.
- The monitor requires 6 hours to warm from the minimum/maximum storage temperature between uses until the monitor is ready for its INTENDED USE when the ambient temperature is 20 C°

**NOTICE – To reduce the risk of product/property damage:**

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling. Protect the unit against hard knocks (e.g. dropping the unit), moisture, water, dirt, dust, chemicals, extreme hot or cold temperatures, major temperature fluctuations, direct exposure to sunlight and heat sources which are too close (e.g. stoves, heating radiators). This may damage the unit.
- The device must be stored in the specified ambient conditions. Please see section 'Care, Maintenance and Disposal' and section 'Technical Specification' for details.
- Do not wash the unit in a washing machine, dish washer or tumble dryer.

- Observe the local regulations for material disposal. Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
- Never attempt to repair, open and/or disassemble the unit (including wrist cuff and optional accessory) or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.
- Do not drop or insert any object into any opening. This may damage the unit.
- Do not press the buttons with excessive force or with pointed objects.
- To conserve the batteries, the monitor switches off automatically if no buttons are pressed for 3 minutes.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

- If you are using the unit for the first time, please remove the protective film from the display.
- Further documents which include information for the Calibration of the device can be asked at the service address.
- The device can be tested in case of supposed inaccuracy every two years. Please contact the nearest service address.

### **Notes On Electromagnetic Compatibility**

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions

or a decrease in the devices electromagnetic immunity; this can result in faulty operation.

- Failure to comply with the above can impair the performance of the device.

### **Battery Handling Safety Precautions**

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.

- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.
- If your skin and eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- Do not disassemble, split or crush the batteries.

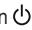
## 2. Getting to Know your Instrument

This wrist blood pressure monitor performs non-invasive measurement and monitoring of adult arterial blood pressure. You can use it to measure your blood pressure quickly and easily, store the results, and display stored readings and their average. This device also features an irregular heartbeat indicator, with values classified and graphically evaluated according to industry guidelines.

## 3. Package Contents

- 1 x Wrist Blood Pressure Monitor BC 21
- 1 x Wrist Cuff for wrist circumferences 5.5 in – 7.7 in (14 – 19.5 cm)
- 2 x 1.5 V LR03 AAA batteries
- 1 x Storage Box
- 1 x Instruction manual



## 4. Parts and Controls

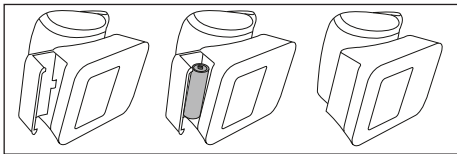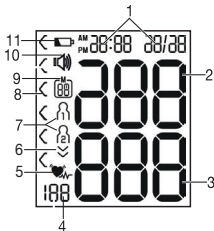1. Memory button **M**
2. Start/stop button 
3. Wrist cuff
4. LCD Display
5. Battery compartment (on rear)
6. Blood Pressure Classification Scale

## LCD Display Information:

1. Time and date
2. Systolic pressure
3. Diastolic pressure
4. Pulse Rate
5. Pulse Icon  / Irregular Heartbeat Indicator 
6. Inflation, Deflation Indicator 
7. User Number  / 
8. Blood Pressure Classification 
9. Memory number / Display for average (A) / AM/PM Values
10. Voice Icon 
11. Battery Level Indicator 

When the Low Battery Icon flashes, replace the batteries. After inserting batteries, the 12h time format indicator flashes on the LCD Display and you can then set the date and time.

### Setting the hour format, date, and time:

If date and time are not set, stored measurements will not have a valid timestamp associated with them.

1. If the time format is not flashing in the LCD Display, press and hold the Memory Button **M** for about five seconds.
2. With the time format flashing on the LCD Display, press the Memory Button **M** until the desired time format appears (12 for AM/PM format or 24 for 24-hour format) and then press the **START/STOP** button  to confirm.   

3. The year flashes on the LCD Display.   
Press the Memory Button **M** to set

## 5. Preparing for Measurement

### Inserting/Replacing Batteries:

1. Remove the Battery Compartment Cover.
2. Insert two AAA size alkaline batteries (included) according to the polarity markings inside the battery compartment. Do not use rechargeable batteries.
3. Reattach the Battery Compartment Cover.

the year and then press the **START/STOP** button  to confirm.

4. The month flashes on the LCD Display. Press the Memory Button **M** to set the month and then press the **START/STOP** button  to confirm.

5. The day flashes on the LCD Display. Press the Memory Button **M** to set the day and then press the **START/STOP** button  to confirm. 

6. The hour flashes on the LCD Display. Press the Memory Button **M** to set the hour and then press the **START/STOP** button  to confirm. 

7. The minutes flash on the LCD Display. Press the Memory Button **M** to set the minutes and then press the **START/STOP** button  to confirm. 

8. **L 1** will flash on the LCD Display and the unit will announce 'English'. Press and hold the Memory Button **M** to select a language; when the preferred language is announced, press the Memory Button **M** to confirm.

L1 = English

L3 = Spanish

L2 = French

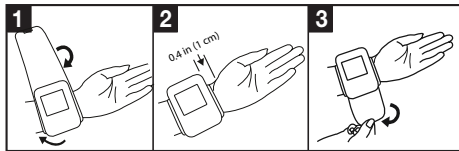
L0 = No audio output

9. “**1 3**” will flash on the LCD Display to indicate the volume level. To lower or shut off the audio response, press and hold the Memory Button **M** to select a volume level (off - A3), then press the **ON/OFF** Button  to exit; the unit will also shut off by itself after a few seconds if no buttons are pressed.

## 6. Measuring Blood Pressure

Keep the blood pressure monitor at room temperature when measuring. Measurements can be performed on either wrist.

### Attaching the cuff

1. Remove clothing and jewelry from your lower arm. Ensure that blood circulation in the arm is not restricted by tight clothing or similar items.
2. Place the unit on the inside of your wrist with your palm facing upwards.

3. Fasten the cuff with the hook-and-loop strip so that the upper edge of the device is positioned approx. ½ inch (~1 cm) below the bottom of your hand. The cuff must fit snugly around your wrist but must not constrict blood flow.

Blood pressure can vary between the right and left arm, which may mean that measured blood pressure values are different. Always perform the measurement on the same arm. If the values between the two arms are significantly different, consult your doctor to determine which arm should be used for the measurement.

#### Correct Posture:

- Rest for at least five minutes before each measurement.
- Perform the measurement either sitting or lying down. Always make sure that the cuff is level with your heart.
- To perform a measurement, sit comfortably with your arms and back supported. Place your feet flat on the floor and do not cross your legs.
- Keep still during the measurement and do not talk.

#### Performing a Measurement:

1. With the unit attached properly and in a correct posture, press the Start/Stop Button. All LCD Display elements will briefly appear and measurement will then begin after about three seconds. To cancel a measurement at any time, press the **START/STOP** Button.
2. As a measurement concludes, air pressure in the Cuff will release. The pulse symbol (♥) and pulse rate will display, along with systolic and diastolic pressure.
3. If a measurement is not performed properly, „Er“ and a number will appear. Refer to the „Troubleshooting Guide“ section for details about specific error messages and then repeat the measurement after you have resolved.
4. To store a measurement, select a user memory by pressing the Memory Button **M** until the desired user number displays. If no user memory is selected, the measurement is stored in the most recently used memory number.
5. Press the **START/STOP** Button to shut off the blood pressure monitor. The device will also shut itself off after about one minute.
6. Wait at least one minute before taking another measurement.

## 7. Evaluating Results

### Irregular Heartbeat:

This device can detect irregular heartbeat patterns during measurement and will indicate them by displaying the irregular heartbeat indicator .

This may be an indicator for arrhythmia, which is a condition where the heart rhythm is abnormal. These symptoms (omitted or premature heartbeats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress, or lack of sleep. Arrhythmia can only be ascertained through examination by your doctor.

Repeat the measurement if the Irregular Heartbeat Indicator displays after a measurement. Rest for at least five minutes between measurements and do not talk or move during measurement. If the icon appears often, contact your doctor, since self-diagnosis and treatment based solely on the test results may be dangerous. It is vital to follow your doctor's instructions.

### Risk Classification:

Measurements can be classified and assessed using the table below. However, these standard values are only a general guideline, since individual blood pressure varies in different people and different age groups, etc.

| Hypertension Category          | Systolic (in mmHg) | Diastolic (in mmHg) | Action   |
|--------------------------------|--------------------|---------------------|--|
| Level 3: severe hypertension   | ≥ 180              | ≥ 110               | seek medical advice                                  |
| Level 2: moderate hypertension | 160–179            | 100–109             | seek medical advice                                  |
| Level 1: mild hypertension     | 140–159            | 90–99               | Have your doctor check your blood pressure regularly |
| High normal                    | 130–139            | 85–89               | Have your doctor check your blood pressure regularly |
| Normal                         | 120–129            | 80–84               | Continue self-monitoring                             |

| Hypertension Category | Systolic (in mmHg) | Diastolic (in mmHg) | Action                   |
|-----------------------|--------------------|---------------------|--------------------------|
| Optimal               | <120               | <80                 | Continue self-monitoring |

Source: WHO, 1999 (World Health Organization)

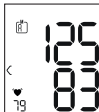
It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the systolic and diastolic values fall into two different WHO categories, the WHO classification Indicator on the device always shows the higher category.


## 8. Displaying and Deleting Measurements

Every successful measurement is stored along with its date and time. If there are more than 60 measurements, the oldest measurement is deleted.

Press the Memory Button **M**; **A** will display along with the average value of all saved measurements for the active user number. Press the Memory Button



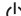
**M** again; **AM** will appear on the LCD Display along with the average morning (5 AM - 9 AM) measurements for the last seven days. Press the Memory Button **M** a third time; **PM** will appear on the LCD Display along with the average evening (6 PM - 8 PM) measurements for the last seven days.

Press the Memory Button **M** again to display the last measurement taken. Keep pressing the Memory Button **M** to display earlier measurements. To shut the device off, press the **START/STOP** button .

### Deleting Stored Values:

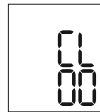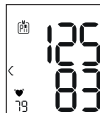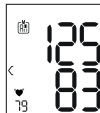To delete measurements, you must first select a user memory. Press the Memory Button **M** until **A** appears in the LCD Display along with the average measured values. Press and hold both the Memory Button **M** and the **START/STOP** button  for five seconds; all stored measurements for the active user number will be deleted. “**CL**” and “**00**” will appear in the display and the device will then shut off.

## 9. Care, Maintenance and Disposal

- Clean the device and cuff with a slightly moistened cloth.
- Do not use detergents or solvents.
- Never immerse the device in water or other liquids.
- When storing the device, make sure no heavy objects are placed on top of it. Remove the batteries.
- Not servicing/maintenance while the monitor is in use.

### Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 10. Troubleshooting Guide

In case of errors,  $E_r$ \_ will appear in the LCD Display, where „\_“ is a number or letter. Error messages may appear if:

- Systolic or diastolic pressure could not be measured ( $E_r 1$  or  $E_r 2$ )
- Systolic or diastolic pressure is outside measurement ranges (Hi or Lo)

- The cuff is too tight or too loose ( $E_r 3$  or  $E_r 4$ )
- Pressure is over 300 mmHg ( $E_r 5$ )
- Measurement takes longer than three minutes ( $E_r 6$ )
- Another system or unit error occurs ( $E_r R$ ,  $E_r D$ ,  $E_r 7$  or  $E_r 8$ )
- Batteries are weak or depleted (  )

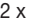
In such cases, repeat the measurement. Do not move or speak. If necessary, replace the batteries.

### Alarm:

If any part of the blood pressure measurement is outside measurement limits, the LCD Display will show Hi or Lo as appropriate. In such cases, you should seek medical assistance and check the accuracy of your procedure. Measurement limits are factory set and cannot be adjusted or deactivated. The message on the LCD Display will disappear after about eight seconds.

## 11. Technical Specification

|                    |   |
|--------------------|---|
| Model no.          | BC 21   |
| Measurement method | Oscillometric, non-invasive blood pressure measurement on the wrist |

|                        |   |
|------------------------|---|
| Measurement range      | Cuff pressure 0–300 mmHg,<br>Systolic 60 -260 mmHg,<br>Diastolic 40 -199 mmHg,<br>Pulse 40–180 beats/minute |
| Display accuracy       | Systolic $\pm 3$ mmHg,<br>Diastolic $\pm 3$ mmHg,<br>Pulse $\pm 5\%$  |
| Measurement inaccuracy | Max. permissible standard deviation according to clinical testing:<br>Systolic 8 mmHg/<br>Diastolic 8 mmHg  |
| Memory                 | 2 x 60 memory spaces  |
| Dimensions             | 3.4 in L x 2.6 in W x 1.2 in H<br>(87 mm L x 67 mm W x 31 mm H)   |
| Weight                 | 3.7 oz (105g) (without batteries)   |
| Cuff size              | 5.5 in - 7.7 in (14 - 19.5 cm)  |
| Operating Range        | 50° F - 104° F (10° C - 40° C),<br>$\leq 85\%$ relative humidity (non-condensing)                           |
| Storage Range          | -4° F - 122° F (-20° C - 50° C),<br>$\leq 85\%$ relative humidity   |
| Power supply           | 2 x 1.5V  AAA batteries    |

|              |                            |
|--------------|----------------------------|
| Battery life | Approx. 200 measurements   |
| Service life | 3 Years/10000 Measurements |

The serial number is located on the device or in the battery compartment.

Technical information is subject to change without notification to allow for updates.

- This device complies with European Standard EN60601-1-2 (in accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8) and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this device.
- The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktgesetz) and the standards EN1060-1 (Non-invasive sphygmomanometers – Part 1: General requirements), EN1060-3 (Noninvasive sphygmomanometers – Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC80601-2-30 (Medical electrical equipment – Part 2 – 30: Particular requirements for the basic safety and essential performance of automated

non-invasive sphygmomanometers). The device complies with the ISO 81060-2 2013 (Non-Invasive Sphygmomanometers - Part 2: Clinical Validation Of Automated Measurement Type).

- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If the device is used for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

## 12. FCC Compliance Information

Wrist Blood Pressure Monitor BC 21

Responsible Party – U.S. Contact Information

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

### FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## 13. Warranty

### Limited Lifetime Warranty For Original Purchaser

Your Beurer Wrist Blood Pressure Monitor, model BC 21, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Wrist Blood Pressure Monitor, model BC 21, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or

repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION**

**OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.**

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or

guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: [www.beurer.com](http://www.beurer.com)

For Canada only:

Manufacturer:

Beurer GmbH, Söflinger Str. 218  
89077 Ulm, Germany

Distributed by:

Beurer North America LP  
1 Oakwood Boulevard, Suite 255  
Hollywood, FL 33020, USA  
[www.beurer.com](http://www.beurer.com)

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

Made in China

## 14. ELECTROMAGNETIC COMPATIBILITY INFORMATION

**Table 1 - Emission**

| Phenomenon                       | Phenomenon                   | Electromagnetic environment |
|----------------------------------|------------------------------|-----------------------------|
| RF emissions                     | CISPR 11<br>Group 1, Class B | Home healthcare environment |
| Harmonic distortion              | IEC 61000-3-2<br>Class A     | Home healthcare environment |
| Voltage fluctuations and flicker | IEC 61000-3-3<br>Compliance  | Home healthcare environment |

**Table 2 - Enclosure Port**

| Phenomenon              | Basic EMC standard | Immunity test levels                            |
|-------------------------|--------------------|---|
|                         |                    | Home Healthcare Environment                     |
| Electrostatic Discharge | IEC 61000-4-2      | ±8 kV contact<br>±2kV, ±4kV, ±8kV,<br>±15kV air |

| Phenomenon   | Basic EMC standard | Immunity test levels                    |
|--|--------------------|---|
|  |                    | Home Healthcare Environment             |
| Radiated RF EM field                                       | IEC 61000-4-3      | 10V/m<br>80MHz-2.7GHz<br>80% AM at 1kHz |
| Proximity fields from RF wireless communications equipment | IEC 61000-4-3      | Refer to table 3                        |
| Rated power frequency magnetic fields                      | IEC 61000-4-8      | 30A/m<br>50Hz or 60Hz                   |

**Table 3 – Proximity fields from RF wireless communications equipment**

| Test frequency (MHz) | Band (MHz) | Immunity test levels                         |
|----------------------|------------|--|
|                      |            | Professional healthcare facility environment |
| 385                  | 380-390    | Pulse modulation 18Hz, 27V/m                 |

| Test frequency (MHz) | Band (MHz) | Immunity test levels                         |
|----------------------|------------|--|
|                      |            | Professional healthcare facility environment |
| 450                  | 430-470    | FM, $\pm 5$ kHz deviation, 1kHz sine, 28V/m  |
| 710                  | 704-787    | Pulse modulation 217Hz, 9V/m                 |
| 745                  |            |  |
| 780                  |            |  |
| 810                  | 900-960    | Pulse modulation 217Hz, 9V/m                 |
| 870                  |            |  |
| 930                  |            |  |
| 1720                 | 1700-1990  | Pulse modulation 217Hz, 28V/m                |
| 1845                 |            |  |
| 1970                 |            |  |
| 2450                 | 2400-2570  | Pulse modulation 217Hz, 28V/m                |

| Test frequency (MHz) | Band (MHz) | Immunity test levels                         |
|----------------------|------------|--|
|                      |            | Professional healthcare facility environment |
| 5240                 | 5100-5800  | Pulse modulation 217Hz,<br>9V/m              |
| 5500                 |            |  |
| 5785                 |            |  |

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