


N95 Particulate Respirator

Read Instructions Before Use

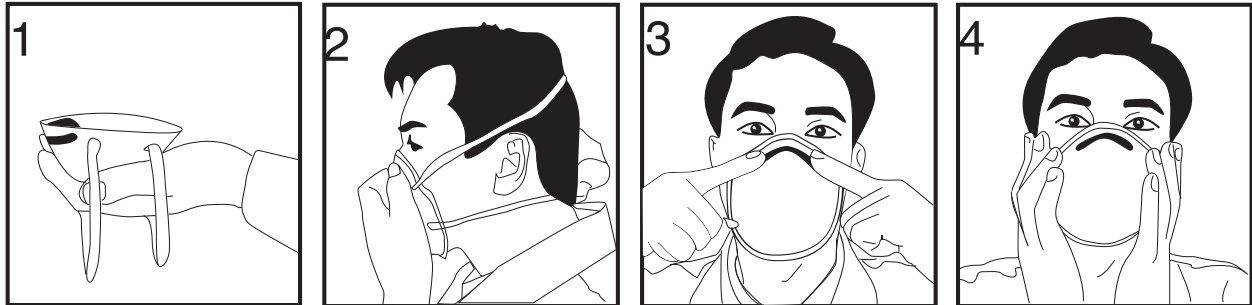

Fitting Instructions:

- 1.** Hold respirator in hand with nose clip at your fingertips and let the straps hang downwards below hand.
- 2.** Place the respirator under the chin and its nose clip upwards. Pull the upper strap to the top back of your head and then pull the shorter lower strap to the back of neck, below the ears.
- 3.** Push the nosepiece inwards with your fingertips to conform the shape of your nose.
- 4.** Face fit check must be done each time the respirator is worn as follows:
 - i. Cover the respirator with both hands, breathing sharply, If leakage along the nose bridge is detected, repeat step 3 and adjust the shape of clip to the nose bridge.
 - ii. If leakage is detected in the rest of respirator edge, adjust the straps again.
 - iii. If leakage cannot be eliminated, do not wear the respirator and stop going into the contaminated environment and turn to the supervisor for help.